

# LipoClear® Plus

Reagent for clearing lipemic serum or plasma.

## Safe-Effective Process

- Inexpensive and convenient
- Non toxic, non-carcinogenic reagent, assures safe handling
- Most common analytes are not affected including enzymes and bilirubin



## Available in convenient 30 mL bottle

- More cost effective for high volume labs
- Can clear any volume of serum or plasma

## Simple Procedure

- Add lipemic sample according to recommended procedure
- Add LipoClear Plus reagent
- Mix well and let stand for 5 minutes
- Centrifuge and remove clarified sample for analysis
  - Multiply results by 1.2



LipoClear Plus can be used with the StatSpin® Express 2 or any conventional centrifuge

**StatSpin®**

PRODUCTIVITY TOOLS.



## StatSpin® Express 2

- StatSpin Express 2 separates lipemic serum or plasma in 180 seconds at 8,500 rpms. It will accept up to 13x 75mm tubes. Other centrifuges can require 20 minutes or more.
- Whisper Quiet: generates only about 60dB of noise
- Simple: Load tubes and press start. Run at the "Hard Spin" setting.

### Convenient low cost method

LipoClear Plus reagent when used with StatSpin Express 2 can clear serum or plasma in 180 seconds

#### Product #

SSX2

#### Description

Express 2 100-240V 50/60Hz

Includes 4 place rotor, universal power supply, removable chamber liner, four 5 mL inserts and North American power cord.

### LipoClear Ordering Information

Product #	Volume	Ratio
LCP1	30 mL	1 part LipoClear Plus to 5 parts patient sample

**StatSpin®**

PRODUCTIVITY TOOLS.

85 Morse Street  
Norwood, MA 02062  
Tel) 781.551.0100  
Fax) 781.551.0036  
800-782-8774  
www.statspin.com